



ULTRA-GRIP GRAB BAR

User Manual

Read this manual carefully prior to use. Retain this manual for further reference.

WARNING:

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. Medic Therapeutics shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.

- Be Advised:
 - Balance assist
 - Always make sure the grab bar is securely attached before each use and always reattach if loose.
 - Only mount to smooth, flat, non-porous surface.
 - Clean and dry both suction cups and mounting surface before first use.
 - Re-attach the grab bar and check the safety indicators before each use.
- DO NOT use for leverage or to support body weight.

PRODUCT INSTALLATION:

1. Select a smooth, flat, non-porous mounting surface with a completely smooth 3.5" diameter area for both suction sections. Clean the surface with a damp cloth and dry thoroughly.
2. Release suction levers by pulling them up and away from the handle.
3. Firmly press the suction cups onto the mounting area and push the levers down.


IMPORTANT: Be sure the secure suction color indicator fully shows green.

Secure Suction Color Indicators:

- **Green:** Secure and ready for use.
- **Red:** Not secure to the wall or surface.

Note: Test suction strength by pulling on the handle. If loose, repeat steps 1-3 to ensure suction security.

PRODUCT OVERVIEW:

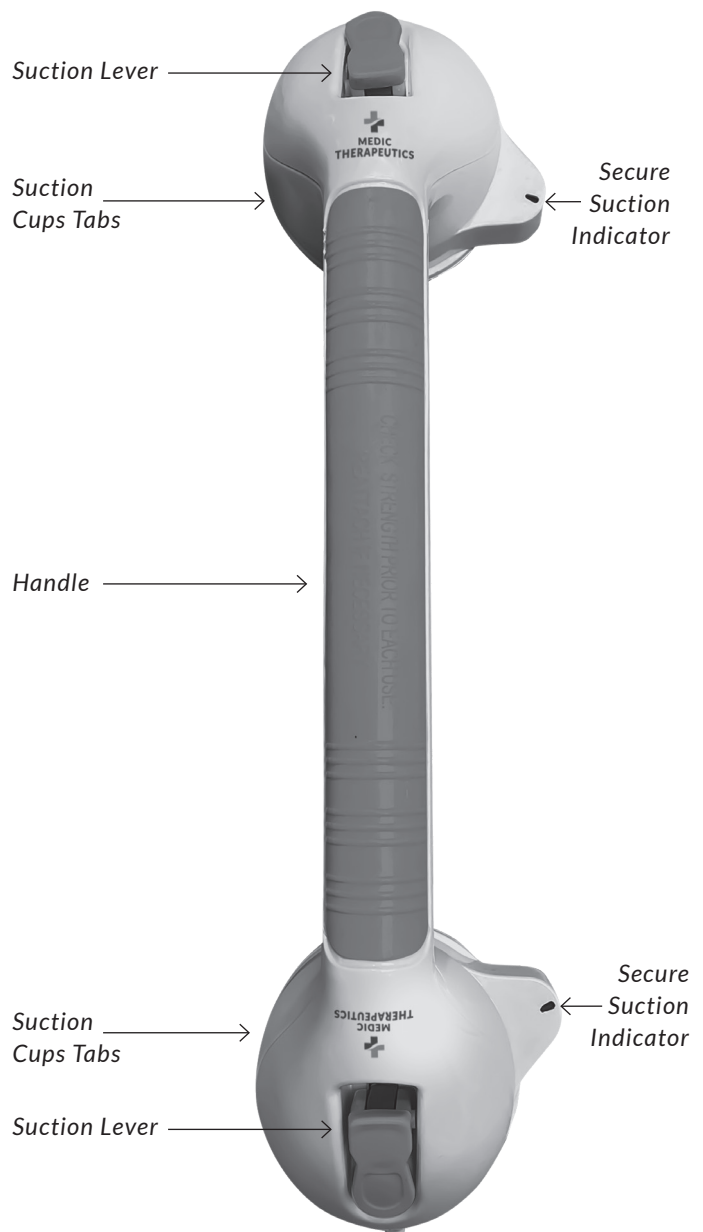

TO REMOVE:

Flip both levers up and lift away from the wall. Use suction cups tabs to help release from mounting surface.

CARE AND MAINTENANCE:

Clean the surface with a damp cloth and dry thoroughly.

LIMITED LIFETIME WARRANTY:

Your Medic Therapeutics product is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics product is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced, we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$35.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount of \$35.00 made out to Medic Therapeutics to:

Address:

Medic Therapeutics Service Center
3069 Taft Street
Hollywood, FL 33021

Contact:

warranty@medictherapeutics.com