



**MEDIC
THERAPEUTICS**

4-IN-1 AUTO ASSIST HANDLE



User Manual

PRODUCT OVERVIEW



WARNING

- Suitable for indoor use only.
- Do not attempt to replace the LED bulbs.
- Keep away from sources of heat.
- Check for damage before each use, if damaged dispose of product in accordance to local regulations.
- This product is not a toy, please keep away from children and pets.
- If you or anyone in your family has ever had symptoms related to epilepsy when exposed to flashing lights, consult your doctor prior to using this device.

- Do not mix old and new batteries.
- Do not mix alkaline and zinc batteries.
- Ensure batteries are installed with the correct polarity symbols.
- Use only suggested battery type.
- Do not use rechargeable batteries.
- Keep batteries out of reach of children.
- Do not short circuit battery terminals.
- Remove batteries if the product is not to be used for a prolonged period of time.
- Never attempt to open a battery.
- Never throw batteries into a fire.
- Remove exhausted batteries.
- Button cell batteries contain mercury. Do not put in trash, recycle or dispose as hazardous waste.
- Button cell batteries should be changed by adults only. Keep away from children.

CHANGING THE BATTERIES & BATTERY WARNING

When the LED flashlight begins to weaken or stops shining altogether, the batteries have been depleted. Follow the steps below to replace the batteries:

- Remove the battery compartment cover on the side of the device using a small screwdriver (not included).
- Insert 2x new batteries and ensure to match with the correct polarity symbol (+/-).

BATTERY SPECIFICATIONS

- Includes 3xAG13(LR44) batteries that power the White LED Bulb
- Battery Life: Approximately 48 hours (continuous use)

PRODUCT SPECIFICATIONS

- Size of product itself: 4*11*18cm
- Net weight of product itself: 162g
- Single package size: 18.7*12.3*7.5cm
- Net weight of product including package: 390g
- Materials: Aluminum alloy + carbon steel + ABS plastic

HOW TO USE (4-IN-1 FUNCTIONS)

1. Car Cane:

Open your car door and snap the tip of your device on. Ensure the car cane is locked into place prior to use (See image).



CAR CANE WARNING

Weight Limit:


The maximum weight limit of this device (when being used as a car cane when entering/exiting the car) is 300 lbs. Do not use if weight exceeds this limit.

IMPORTANT NOTE

The car cane is not expected to scratch your car's door. However, its bottom part might come into contact with your vehicle's paint. There is a chance that with repeated use it may cause some scratching, but a very insignificant amount. Use the product carefully to prevent scratching your car paint.

2. LED Flashlight:

Located on the handle of the device. Press the On/Off button to power the flashlight.

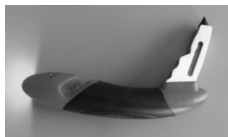

3. Window Breaker:

Located at the top of the device. Smash the corner of the window to break the glass the fastest. Make sure to protect your hand from the glass.



4. Seat Belt Cutter:

Located at the bottom of the device. Insert your seat belt into the device's cavity. Push down and away from you in one sweeping motion to slice the seat belt.



LIMITED LIFETIME WARRANTY

Your Medic Therapeutics product is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics product is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced, we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$5.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount \$5.00 made out to Medic Therapeutics to:

Address:

Medic Therapeutics Service Center
3069 Taft Street
Hollywood, FL 33021

Contact:

warranty@medictherapeutics.com

