

It is important to periodically clean your hearing aids to ensure the optimal use of your device.

- Clean the device over a desk, table, or stable surface prevent dropping and damaging the device.
- Wipe down the overall device with a dry soft cloth to remove dust and unwanted particles.
- Use the included brush to clean and remove earwax from the sound outlet and ear domes.
- Do NOT use water or any liquids to clean, as doing so may permanently damage the device.

**IMPORTANT NOTE:**

Avoid dropping or squashing the hearing aids when inserting and removing them from your ears. Failing to do so may cause damage to the microphone and receiver of the device. Handle with caution.

