



MULTI-PURPOSE USE:



FEATURES:

## MEDIC THERAPEUTICS HOT & COLD COMPRESSION PAIN RELIEF EYE MASK

3-PACK

- Natural Lavender for Aromatherapy Benefits
- Flaxseeds Puts Gentle Pressure on Eyes/  
Sinuses & Retains Temperature for Hot/Cold Therapy

For Rest & Relaxation



**HOT/COLD THERAPY INSTRUCTIONS:**

**HOT**  
Microwave for 30-60 sec (800-1000 watts). If the mask is too hot to the touch, let it cool down prior to use. If it does not feel hot enough, microwave again for 1 additional min.

**COLD**  
Place either in the refrigerator for 1-2 hrs or freezer for 30-45 min. If the mask is too cold, icy or stiff, let it sit for a few minutes and thaw out prior to use.

**CLEANING INSTRUCTIONS:**  
Machine washable cover. Remove the interior bag of contents and set aside. ONLY wash the exterior mask cover and allow to dry before re-inserting the interior bag of contents.

# MEDIC THERAPEUTICS

## HOT & COLD COMPRESSION PAIN RELIEF EYE MASK

### PRODUCT SPECIFICATIONS:

- Size of product itself: 23 cm \* 10 cm
- Net weight of product itself: 170g
- Single package size: 27cm\*15cm
- Net weight of product including package: 510g
- Materials: Exterior fabric + Interior filling  
(95% flaxseed & 5% dried lavender flowers)



### NON-MEDICAL USE

Medic Therapeutics is a trademark name for a family of products that includes a variety of emergency and wellness supplies such as vibrating fitness platform, walking canes, sterilizers, massage guns, thermometers, face coverings and masks (such as coverings of the nose and mouth that may include masks as well as bandanas, scarves, and other cloth coverings). None of Medic Therapeutics products are designed or intended to prevent, diagnose or treat any illness, condition or disease, even when used as directed. This product is not a medical device. This product is not intended for use in the diagnosis of disease or other conditions or in the cure, mitigation, treatment, or prevention of disease. Always consult your doctor or physician for any medical questions you may have.

This product is designed to help promote relaxation. Always consult your doctor or physician immediately for any medical questions you may have. This apparatus is NOT guaranteed as a prevention or treatment for any disease or medical condition. For proper use only. Operate only as directed. Misuse may result in sickness or death. Always carefully review instructions for use in the operating instructions manual prior to use.

Keep out of reach of children and supervise children under the age of eighteen (18) at all times. Do not allow the user to place this item in his/her mouth at any time. No child or minor should ever be left unattended with any of these items, as small parts may become loose & pose a choking hazard.

**WARNING:** Do not use in areas exposed to direct sunlight. Keep away from children at all times.

### LIMITED LIFETIME WARRANTY

Your Medic Therapeutics product is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics product is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced, we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$8.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount of \$8.00 made out to Medic Therapeutics to:

**Address:**  
Medic Therapeutics Service Center  
3069 Taft Street  
Hollywood, FL 33021

**Contact:**  
warranty@medictherapeutics.com

**MT-PAINRMASK-001    MADE IN CHINA**

